

# How to safely clean your reusable face shield



Perform hand hygiene **BEFORE** and **AFTER** cleaning your reusable face shield



Saturate a clean cloth with 70% isopropyl alcohol  
Do not use bleach



Wipe face shield **INSIDE** then **OUTSIDE** with the cloth



Air dry thoroughly in a safe place before storing or using again



Store in a clean zip lock bag or container once dry, when not in use

To improve visibility through your reusable face shield: use water to remove residue or wipe with an alcohol wipe

Face shields are high-risk surfaces, which can become contaminated with coronavirus (COVID-19).

Clean and disinfect your own reusable face shield before and after use.

Avoid touching the front of your face shield or other eye protection.